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**Numero verde consumatori Italia 800 900 080**

## EN ANEROID SPHYGMOMANOMETER WITH STETHOSCOPE

Dear Customer, thank you for purchasing the Pic Solution aneroïd sphygmomanometer, a robust, easy-to-carry product that guarantees maximum reliability for professional use. The aneroïd sphygmomanometer is used to measure systolic and diastolic pressure. In this model the manometer and the inflation ball are two separate items, the former for reading pressure values and the latter for regulating the pressure of the arm cuff; the stethoscope is included meaning that it is a complete kit. All Pic Solution sphygmomanometers undergo stringent quality control checks, in accordance with EU legislation, to guarantee the utmost quality and precision.

**WARNING** Read the instructions carefully before use and keep for future reference.

The sphygmomanometer is designed to measure blood pressure.

### Instructions for use

- Place the **arm cuff (1)** around your right or left arm, making sure that the bottom edge is 1-2 cm above the crook of your elbow. Check that the circumference of your arm corresponds with the limits indicated on the arm cuff and that the arm cuff is parallel with your heart. Apply to unbroken skin (Figure A).
- Close the arm cuff, inserting the end of it in the **metal ring (2)** and attaching it to the velcro so that it is firmly around the arm without being too tight.
- Place the **arm cuff marking (3)** on the artery and insert the head of the stethoscope under the arm cuff so that the membrane is on the internal part of the bicep, next to the artery (Figure B).
- The arm used for the measurement must be kept slightly bent and resting on a surface with the muscles totally relaxed.
- Check that the **deflation valve (4)** on the **inflation ball (5)** is closed, turning it anti-clockwise.
- Pump the inflation ball until the pressure of the arm cuff is no greater than 20-30 mmHg above the maximum expected pressure (on the basis of regular measurements or the personal details or physical condition of the patient).
- Slightly open the deflation valve, turning it clockwise, and allow the air to slowly seep out so the arm cuff deflates at a speed of 2-3mmHg per second (Figure C).
- When the pressure of the arm cuff has gone below the maximum value it is possible to hear the heart beat using the stethoscope. As soon as you hear the first beat, make a note of the reading on the **manometer (6)**, this is the "maximum pressure" (systolic) (Figure D).
- Continue to gradually open the valve until you hear the final beat: the value indicated on the manometer is the "minimum pressure" (diastolic). The procedure described, used for adult patients, involves the identification of diastolic pressure in the fifth Korotkoff phase, K5. To measure diastolic pressure in pregnant women or children it may be appropriate to identify the K4 sound.
- After recording the blood pressure values completely deflate the arm cuff and, if the results of the measurements are not clear, repeat the operation for at least 3-4 minutes.
- After taking the measurements replace the device in the box supplied.

### Here are a few helpful tips to help you obtain more accurate readings:

- It is important that you relax during measurement.
- Take your blood pressure at normal body temperature. If you are feeling cold or hot, wait a while before taking a measurement.
- Relax for 5 minutes before measuring.
- Take the measurement in a silent room.
- Do not move and do not talk when taking a measurement.
- Do not cross your legs and keep your feet on the ground.
- Keep your back against the backrest of the chair.

### IMPORTANT

- In no event inflate the arm cuff above 300mmHg: excess inflation could result in a loss of calibration of the instrument or damage the arm cuff. Taking blood pressure measurements with excessive frequency or over inflating the arm cuff can be harmful for the patient due to blood flow interference.
- The auscultatory method is not appropriate for self-measurement.
- The device is not suitable for use on people whose arm circumference falls outside the range of 24 to 32 cm.
- Keep out of reach of children.
- In the event that measurements need to be taken on pregnant women or individuals with specific diseases (e.g., arrhythmia, obesity), the physician must establish procedures and precautions to be observed during the measurement.
- Protect the device from heavy knocks.
- Avoid damaging rubber parts with sharp or pointed instruments.
- Do not use devices that are not intact or which present problems during the arm cuff inflation phase.
- When the arm cuff is deflated the hand should be positioned within the interval marked by the horizontal line around the zero.
- Do not expose the device to heat sources or excessively low temperatures.
- To guarantee efficient measurements it is advisable to recalibrate the device once every two years or in the

event that it is accidentally dropped. The calibration check must be carried out on instruments calibrated in accordance with national or international samples, by authorised centres.

### Care and maintenance

- We recommend using a soft, clean cloth for cleaning the device.
- Do not use chemical products.
- Do not submerge in liquids.
- Do not carry out heat sterilisations

In case of problems concerning settings, maintenance or use of the device, contact your local distributor or the Italian toll-free number 800 900 080. Don't open or repair the device by yourself.

### Technical specifications

Scale grading: 2mmHg
Measurement accuracy: +/- 3mmHg
Measurement range: 0 - 300 mmHg
Operating conditions: 10°C - 40°C, 20% - 85% RH; 700hPA - 1060hPA
Storage and transportation conditions: -20°C - 70°C, 20% - 85% RH; 700hPA - 1060hPA
Store in a cool, dry place
Accessories: Arm cuff [REF](#) 02009127000000, Stethomed [REF](#) 02021405000000

Observe applicable local laws for disposal.

**WARNING** Report any serious accident occurring in connection with the device to your competent authority and to the manufacturer.

### Flat stethoscope

- The stethoscope is made up of:
  - a flat head, suitable for measuring the heartbeat and, therefore, for measuring blood pressure, thanks to the way it can comfortably be inserted under the arm cuff.
  - a Y tube.
  - a headset.
  - 2 ear tips applied to the headset.

### Important

- Always check that the ear tips are attached to the headset before using the stethoscope.
- Do not expose the device to heat sources or excessively low temperatures.
- Avoid damaging the tube or the membrane with sharp or pointed instruments.
- Do not apply pressure to the membrane.
- We recommend using a soft, clean cloth wet with alcohol for cleaning the stethoscope. Do not use other chemical products. Do not submerge in liquids.

### Warranty

The warranty is guaranteed against any lack of conformity under normal use as specified in the instructions. The warranty does not, therefore, apply to damage caused by improper use, wear or accidents. The arm cuff and parts subject to wear are not covered by the warranty, for the duration of the warranty against lack of conformity, see the specific provisions of applicable national laws in the country of purchase, where present.

<b>MD</b>	Medical Device		Recycling cardboard
	Production date		Recycling cardboard
<b>UDI</b>	Unique identifier of the equipment	<b>REF</b>	Catalogue number
	Fragile, handle with care		Temperature limitations
	Do not use hooks		Humidity limitation
	Keep away from sunlight		Warning!
	Der Grüne Punkt		Please refer to The Instructions for use
	Recyclable packaging	<b>LOT</b>	Lot number
	Recycling plastic		Manufacturer
	Recycling plastic		Keep dry
	Symbol of compliance with Regulation (EU) 2017/745.		

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